

ezGLIDE™

INSTRUCTIONS




1
The EZGlide™ transfer sheet conveniently attaches to your table or gurney with the included velcro straps.




2
Pull the cardboard zipper on the top of the cartridge. You will expose the first transfer sheet.




3
Pull the first sheet out of the cartridge. The next sheet will be ready for use.

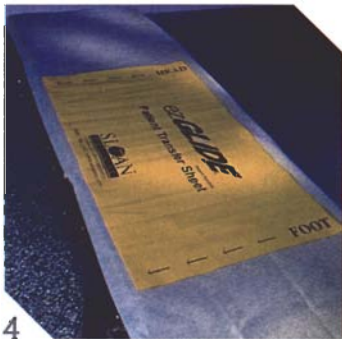

4
Place the EZ Glide™ transfer sheet on top of the table or gurney.

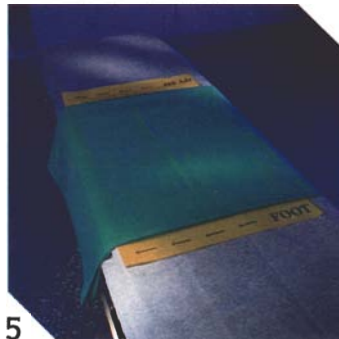


5
Cover EZ Glide™ transfer sheet with draw sheet. Place patient on top of draw sheet.

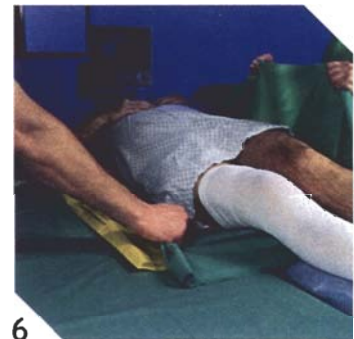

6
Pull linen on which the patient is lying.



7
Continue to pull patient across to destination surface.

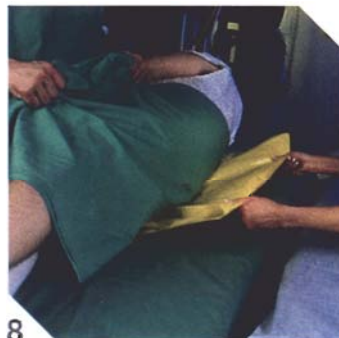

8
Remove EZGlide™ transfer sheet.



9
Discard EZGlide™ transfer sheet.

SLOAN MEDICAL